

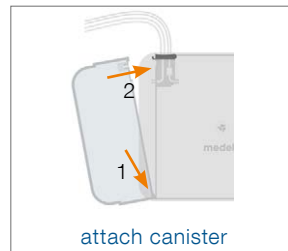


Thopaz™ Quick Card


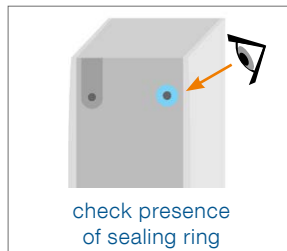


This Quick Card in no way replaces the Instructions for Use REF 200.0685




Prepare Thopaz for use

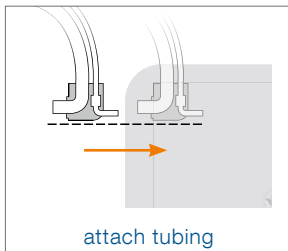




Turn Thopaz on

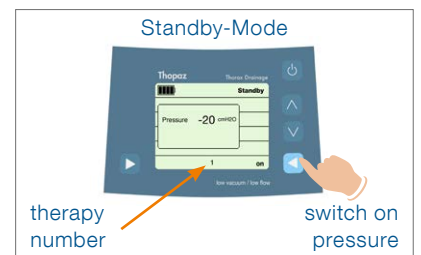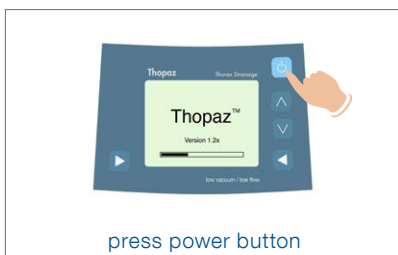




Perform functional check

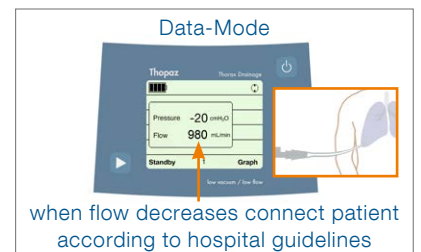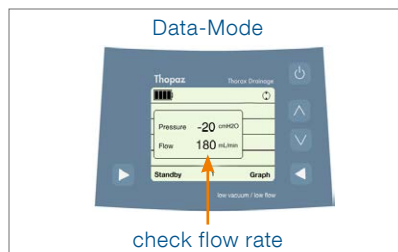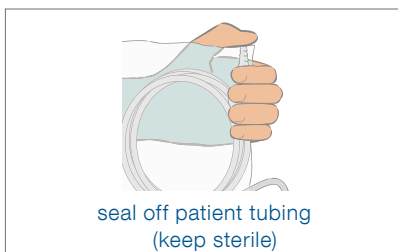







Adjust pressure (when pump is running)

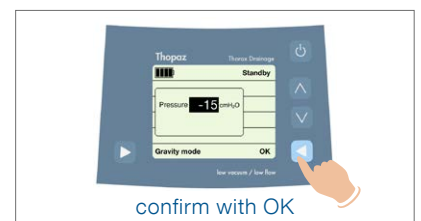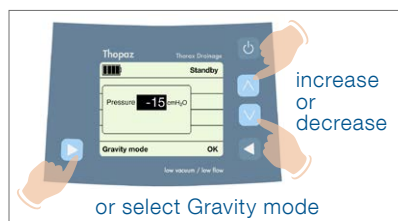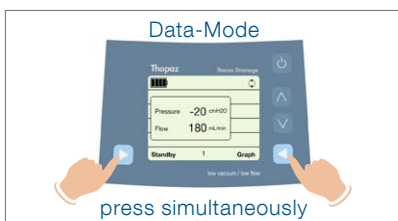

Check therapy progress

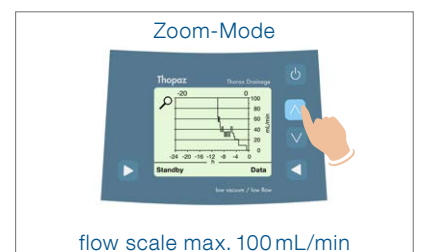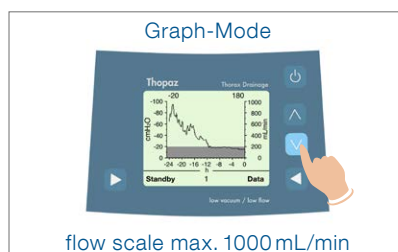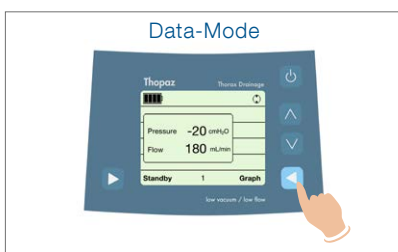

Change canister

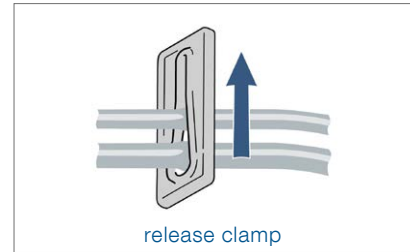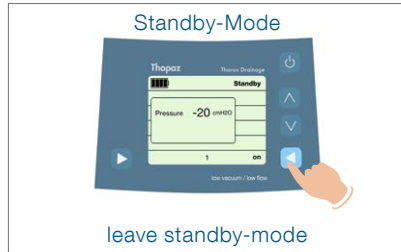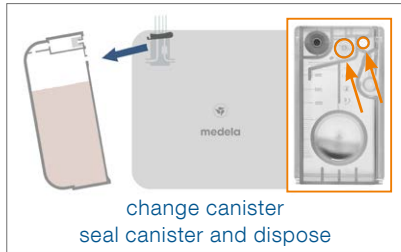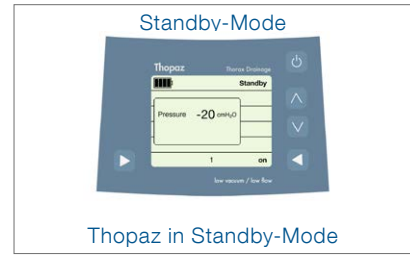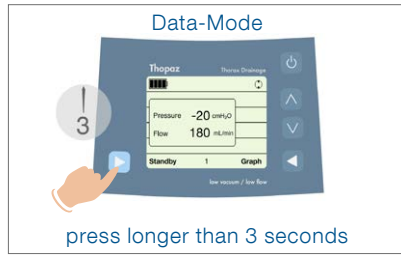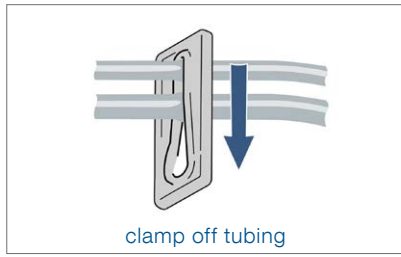

Catheter check




catheter open

catheter clogged


Zoom-Mode
Thopaz Thopaz Drainage
Pressure -20 cmH₂O
Flow 0 mL/min
Standby 1 Data
low vacuum / low flow

- Only visible in Zoom-Mode when flow is 0 mL/min
- Patient must inhale and exhale deeply and slowly

Alarm management

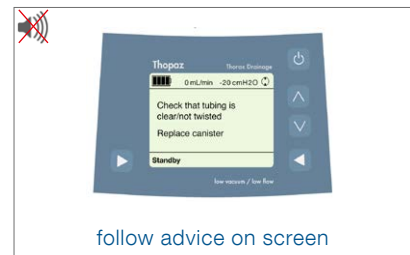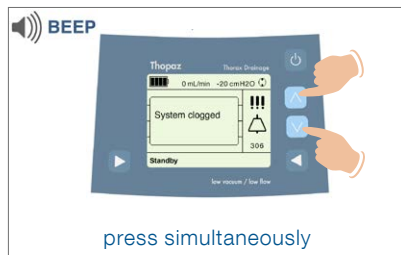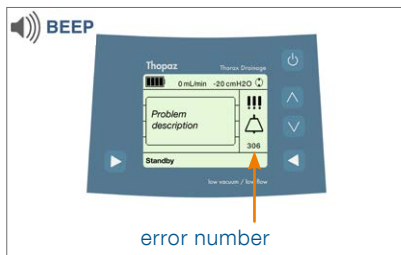

Switch off Thopaz

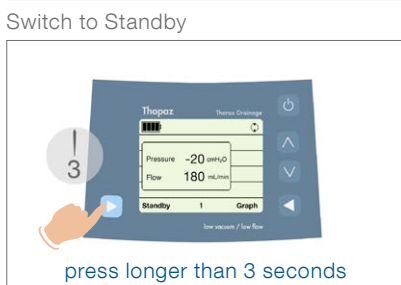

Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
www.medela.com

International Sales
Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
Phone +41 41 562 51 51
Fax +41 41 562 51 00
ism@medela.ch
www.medela.com

CAUTION:
Federal US law restricts this device to sale by or on the order of a physician.