



PROVIDING ADVANCED TREATMENT WITH EASE



Thopaz™

Functionality and simplicity with objective readings

Thopaz sets new standards in thoracic drainage therapy: The compact system provides regulated negative pressure close to the patient's chest, based on the patient's air leak. It monitors the air leak and only applies the suction required to maintain the negative pressure prescribed and set by the managing physician. The digital display provides objective data in real time as well as a 24-hour historical graph of the air leak, which allows easy tracking of the therapy progress. Set-up and handling of Thopaz is very easy, with well-designed accessories adding even more comfort to the lightweight system.

Thopaz offers a progressive, innovative therapy that allows physicians and nursing staff to monitor air leaks objectively, aiding them to make timely, objective decisions in chest tube management – which potentially translates into cost savings for hospitals.



Benefits

- Unrestricted mobility of the patient may promote a quicker healing process.
- Objective real time readings and historical air leak data provide reliable information for timely decision on when to remove the chest tube.
- Important monitoring functions provide peace of mind.
- Easy set-up and operation provides simplicity and ease of use for clinicians.

Early patient mobilization facilitates quicker patient recovery after surgery

Thopaz is lightweight and compact. With an integrated suction source and rechargeable battery, patients enjoy freedom to ambulate without being attached to wall suction. Early ambulation may contribute to a quicker healing process, reducing the length of hospital stay.




Handy and lightweight system

Objective data enables confident decision making for chest tube management

Thopaz provides digital real time data and a 24-hour trend of the air leak. Treatment data of pressure and flow rate can be transferred to a PC and included in the patient's records. Continuously recorded treatment data allows the physician to manage air leaks in an objective manner, providing evidence based decisions for safe chest tube removal. This feature may lead to a reduced number of chest x-rays and an early discharge of a patient.

Important monitoring functions provide peace of mind

The safety features of Thopaz include alarms that alert clinical personnel for prompt intervention in case of irregularities. Thopaz continually monitors and regulates the negative pressure close to the patient's chest, resolving the common problems experienced with the tubing in non-regulated chest drains, such as siphons, clogs and kinks. This also enables the placement of Thopaz at the level of the patient's chest.

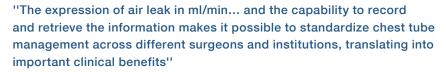

Consensus definitions to promote an evidence-based approach to management of the pleural space. A collaborative proposal by ESTS, AATS, STS and GTSC. Brunelli et al. (2011). Eur J Cardiothorac. Surg.

Ease-of-use supports the clinical personnel in their daily work

The simple assembly of the Thopaz tubing and canister, combined with the closed collection of fluids provides staff with an easy to use, hygienic drainage system. The intuitive functionality makes Thopaz simple to operate, and the display of objective, recorded data means clinicians no longer have to estimate the size of air leaks. Together, these features reduce and simplify the work of the clinical personnel.



Objective data about the therapy



Easy change of canister

Disposables

Thopaz tubings

The system's dual lumen tubing provides continuous monitoring of pressure and evacuation of fluids and air. Two types of connectors are available (single or double) in small, medium or large sizes to accommodate preferred chest tubes. All Thopaz tubing connectors contain a sample port for convenient sampling of drainage fluid.



Thopaz canisters

For the collection of fluids, Thopaz offers three disposable canisters with 0.3, 0.8 and 2 liters capacity and with or without integrated solidifier. The innovative press & shake technology permits an on-demand release of solidifier and an unimpaired fluid quantification until the canister is full.



Accessories

Docking station

Thopaz comes with a docking station that can be placed on a level surface or be attached to a standard rail or IV pole by means of a clamp. More mobile patients can use the carrying strap that is conveniently placed across the shoulder, leaving their hands free.



Rail holder

A universal rail holder is available that is used to attach Thopaz to an IV pole or bed rail. This helps facilitate patient transport and mobility as well as providing extra security for the device while in use.



Technical data



Low vacuum
– 10 kPa
– 75 mmHg
– 100 mbar/cmH₂O



1 kg Without canister 2,2 lbs



low flow 5 l/min.











The unique press & shake technology provides a hygienic and cost effective disposal of canisters.

CAUTION: Federal law restricts this device to sale by or on the order of a health professional.

Local contact:

Medical Vacuum Technology for Healthcare Professionals

Please contact us or your local Medela representative for details.



Medela AG Lättichstrasse 4b 6341 Baar, Switzerland www.medela.com

Australia

Medela Pty Ltd.
Medical Technology
3 Arco Lane
Heatherton, Vic, 3202
Australia
Phone +61 3 9552 8600
Fax +61 3 9552 8699
contact@medela.com.au
www.medela.com.au

Canada

Medela Inc.
4160 Sladeview Cres., Unit #8
Mississauga, ON
L5L 0A1
Canada
Phone +01 905 608 7272
Fax +01 905 608 8720
info@medela.ca
www.medela.ca

India

www.medela.in

Medela India private limited c/o Vatika Business Park First floor, tower 2, Sohna Road, Sec-49, Gurgaon 122 002 India Phone +91 124 4416999 Fax +91 124 4416990 info@medela.in

USA

Medela Inc.
P.O. Box 660
1101 Corporate Drive
McHenry, IL 60050
USA
Phone +1 877 735 1626
Fax +1 815 307 8942
suction@medela.com
www.medelasuction.com

UK

Medela UK Ltd. Huntsman Drive Northbank Industrial Park Irlam, Manchester M44 5EG UK Phone +44 870 950 5994 Fax +44 870 389 2233 info@medela.co.uk www.medela.co.uk